

## TO TRAVEL WITH ECO ASSIST

### WHAT APPLIES?

If, due to functional variation, you need to bring an electric aid with you on the trip, you may transport this as checked baggage.

Please be Informed that, each airline may interpret the regulations themselves or apply their own rules. You must therefore always check yourself with the specific airline about what applies when you book your ticket.

### PLEASE NOTICE

- All batteries brought must be approved for flight. All our aviation batteries for ECO ASSIST are tested and approved according to the international standard UN38.3
- The batteries may have a maximum output of 300Wh (300Wh in the aid and 300Wh in reserve). The ECO ASSIST travel battery has a capacity of 278Wh.
- In some cases, the airline wants one or both batteries to be removed (one counts as a spare battery). The battery(s) you bring as hand luggage must be protected against short circuits. You can solve this by taping over the contacts.

### HOW TO DO

1. When booking a flight, you should always inform the airline in advance that you will be taking an electrical aid with you and follow their instructions. Give the airline detailed information about this in connection with the booking.

2. When flying, the travel batteries can be used. They have the following specifications:

- |                  |                             |
|------------------|-----------------------------|
| • Name           | Airsafe battery ECO ASSIST  |
| • Article number | EBB0048TIA                  |
| • Voltage        | 48 V                        |
| • Capacity       | 5,8Ah, (278 Wh)             |
| • Type           | Lithium                     |
| • Weight         | 1900 g                      |
| • Size           | L:320 mm, H: 87 mm, W: 88mm |
| • Approval       | UN38.3                      |

3. When you check in, the assistance staff will provide you with a manual wheelchair if you do not use your own. You may then bring your ECO ASSIST to special baggage check-in. You should definitely have your ECO ASSIST packed in a sturdy bag, there are various options for prams to buy. Another option is to rent a bag from Air Shells [www.airshells.com](http://www.airshells.com)