

# SPACE LX / SPACE CF


## SPARE PARTS


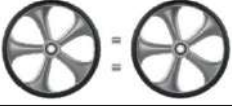








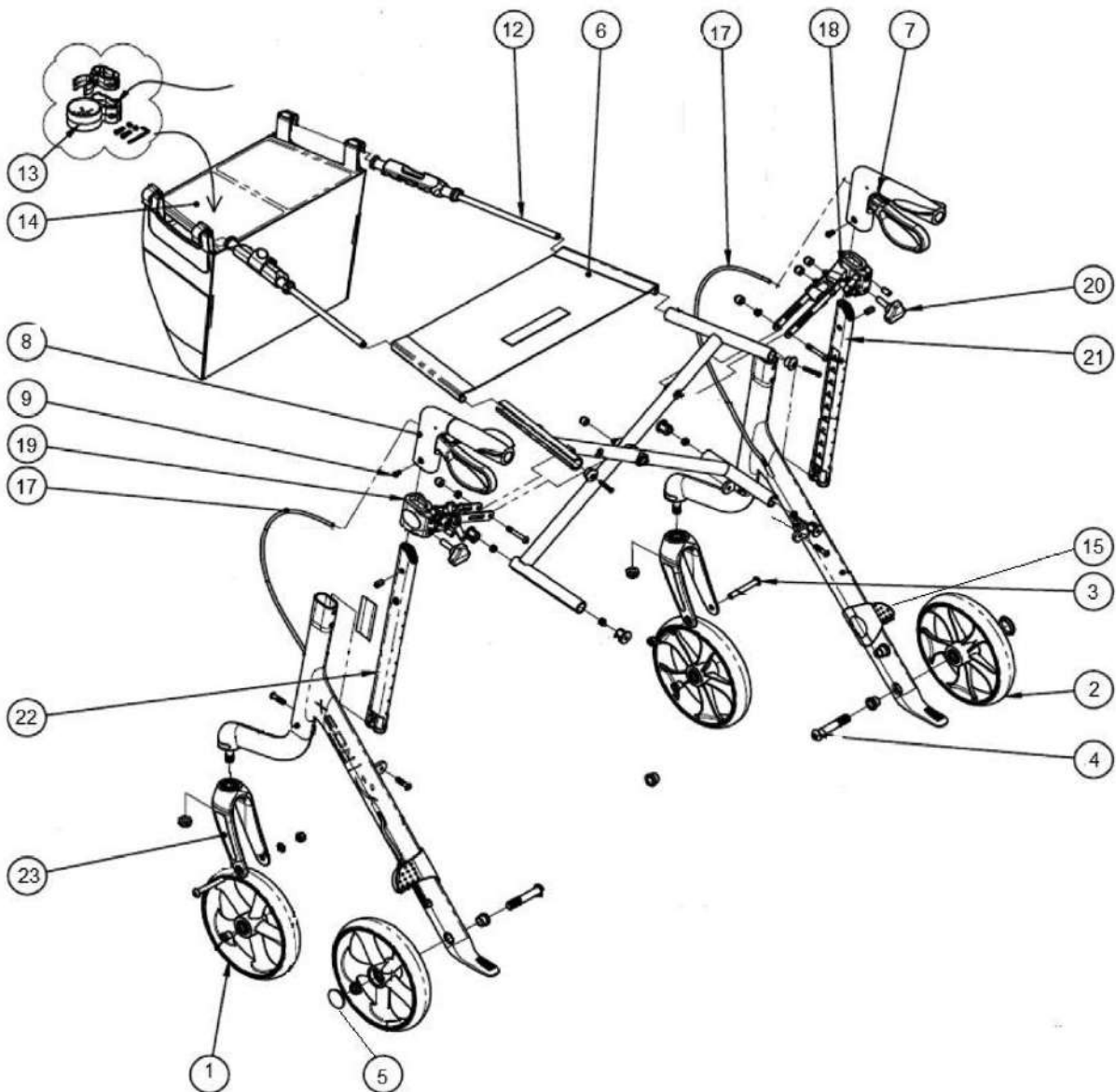
















Pricelist is valid from 1.01.2025 until the new one is published.



Please send orders by email to the following address: [ukadmin@rehasense.com](mailto:ukadmin@rehasense.com)

# TECHNICAL DRAWING FOR SPACE LX / SPACE CF



No.	Picture	Item description	Item no	Trade	RRP
1		Black-silver front wheel, 2pcs/set	ROL30H-RS		
2		Black-silver rear wheel, 2pcs/set	ROL30J-RS		
1		Front wheel with soft <b>black PU tyre</b> and <b>silver rim</b> , (set of 2)	ACR1016FSB		
2		Rear wheel with soft <b>black PU tyre</b> and <b>silver rim</b> , (set of 2)	ACR1016RSB		
3		Fitting set for front wheels of rollator, 1set packed/rollator	1910H7-RS		
4		Fitting set for rear wheels of rollator, 1set packed/rollator	1910CX-RS		
5		Push button d29 x d32.5 x 5 mm, black ethylene vinyl acetate - for hub	641005		
6		Seat incl. strap, 46 cm, black nylon, size <b>L</b>	ROL283-RS		
		Seat incl. strap, 46 cm, black nylon, size <b>M</b>	ROL289-RS		
7		Complete push handle incl. brake lever - <b>Right</b>	ROL340R-RS		
8		Complete push handle incl. brake lever - <b>Left</b>	ROL340L-RS		
9		Screw m 5 x 10 mm, black galvanized - used for fixing the complete push handle ISO 7380, hardness 8.8, tuf-lock.	19102C		
10		<b>Right</b> soft handgrip, black TPR - for rollator, spare level for RSE	ROL11U-RS		
11		<b>Left</b> soft handgrip, black TPR - for rollator, spare part level	ROL11V-RS		
12		Complete steel shaft with a+lock for bag and seat, (set of 2 pcs, <b>Left</b> and <b>Right</b> tubes)	ROL20A-RS		

No.	Picture	Item description	Item no	Trade	RRP
13		Standard crutch holder, black	ACR01043		
14		Basket - new type, for rollator with folding locker system, size L	ROL1AA-RS		
15		Brake block set for rollator, Left + Right side as one set	ROL274-RS		
15		Spring d1.0 x d7.36 (outer diameter) x 45.6 mm, spring steel, galvanized surface.	64100X		
17		Brake cable, size L	69103H-RS		
		Brake cable, size M	69103Y		
18		Compl. locking mechanism for cross for std. rollator L size	ROL341L-RS		
		Compl. locking mechanism for cross for std. rollator M size	ROL34L-RS		
19		Right regulator for Space L size	ROL341R-RS		
		Right regulator for Space M size	ROL34R-RS		
20		Plastic handle knob, black pp with steel m8 external thread insert, by moulded	ROL125		
21		Right side inner tube ASM - L size, 1pcs	ROL342R-RS		
		Right side inner tube ASM - M size, 1pcs	ROL36R-RS		
22		Left side inner tube ASM - L size, 1pcs	ROL342L-RS		
		Left side inner tube ASM - M size, 1pcs	ROL36L-RS		
23		Front fork (Right/Left), black (1 pcs)	R019801-RS		
24		Umbrella holder, including screws and tools and alu. parts, for RSE	ROLIDJ		

No.	Picture	Item description	Item no	Trade	RRP
25		Bolt m5 x 15/-0.2mm, ISO 7380, fully threaded, black galvanized steel, hardness 8.8, tuf-lock.	ROL34C		
		Repair kit for rollators contains 19 different small parts used for Athlon, Server, Athlon HD, Server HD, Explorer, Navigator and Navigator Air, Space	ACR01020R		

**NOTE:** We do not offer cross and side frames as spare parts. With good reason we classify such damages as dangerous wand total loss. It means the user needs a new rollator. Safety is the highest priority.



# REHASENSE

**REHASENSE UK**  
**UNIT 5, GUILFORD ROAD**  
**INDUSTRIAL ESTATE**  
**HAYLE, CORNWALL, TR27 4BA**  
**UNITED KINGDOM**  
☎ **+44 (0) 333 220 0704**  
✉ **UKADMIN@REHASENSE.COM**

REHASENSE ITALY SRL  
VIALE DELLE INDUSTRIE 5  
20044 ARESE (MI) ITALY  
☎ +39.02.99 77 08  
✉ INFO.ITALY@REHASENSE.COM

REHASENSE DEUTSCHLAND GMBH  
STERNBERGER STRASSE 12,  
32699 EXTERTAL,  
GERMANY  
☎ +49 2821 590 8587  
✉ INFO@REHASENSE.COM

REHASENSE DENMARK  
GLARMESTERVEJ 16 A, ST. TV.  
8600 SILKEBORG,  
DENMARK  
☎ +49 176 588 788 01  
✉ INFO@REHASENSE.COM

REHASENSE SP. Z O.O.  
SULEJOWSKA 45G  
97-300 PIOTRKÓW TRYBUNALSKI  
POLAND  
☎ +48 880 331 272  
✉ INFO@REHASENSE.COM

REHASENSE BENELUX  
MARKEWEG 81  
8398GM BLESIJKE,  
NETHERLANDS  
☎ +31 653 837 211  
✉ INFO@REHASENSE.NL

REHASENSE SVERIGE AB  
POLHEMSGATAN 28, SE-392 39 KALMAR  
SWEDEN  
☎ +46 722 325 090  
✉ MATS.LAVEBORN@REHASENSE.COM

REHASENSE CHINA  
NO.289, WENGJIAO ROAD  
XINYANG INDUSTRIAL AREA  
361022 HAICANG, XIAMEN,  
CHINA  
☎ +86 592 688 4014  
✉ EVELYN.CHEN@REHASENSE.COM

REHASENSE KOREA  
1F. 338-46, DONGBUSUNHWAN-RO,  
WONJU-SI 26454 GANGWON-DO,  
REPUBLIC OF KOREA  
☎ +82 070 4216 3429  
✉ JONGSUNG.KIM@REHASENSE.COM

REHASENSE NEW ZEALAND  
UNIT 4, 260 APIRANA AVENUE  
GLEN INNES, AUCKLAND, 1072,  
NEW ZEALAND  
☎ +09 555 3111  
✉ NZ-INFO@REHASENSE.COM

REHASENSE MOBILITY CANADA LTD  
137 BUTTERMILL AVENUE  
CONCORD, ONTARIO, L4K 3X5  
CANADA  
☎ +1 289 654 1901  
✉ INFO.CANADA@REHASENSE.COM

REHASENSE USA  
1333 BURR RIDGE PARKWAY, SUITE 200  
BURR RIDGE, IL 60527,  
USA  
☎ +1 630 756 3099  
✉ INFO-USA@REHASENSE.COM

REHASENSE MENA  
QETAF MEDICAL EST  
3599 PRINCE MOHAMMED BIN  
ABDULAZIZ ST,  
SAUD LANGAWI CENTER, AL ANDALUS  
JEDDAH 23326,  
SAUDI ARABIA  
☎ +96 650 664 1993  
✉ SALES-MIDDLE-EAST@REHASENSE.COM

REHASENSE COLOMBIA  
CALLE 171 # 21 A 47  
BARRIO LA URIBE, BOGOTÁ  
COLOMBIA  
☎ +57 3153118652  
✉ IVANUMANA@REHASENSECOLOMBIA.COM

REHASENSE AUSTRALIA PTY LTD  
☎ +61 475 856 207  
✉ WERNER.PETZER@REHASENSE.COM

